



 **Paragard**[®]
T380A
intrauterine copper contraceptive

simple,
honest
pregnancy
prevention™

GET IT.

The #1 prescribed hormone-free IUD with up to 10 years of pregnancy prevention.

Talk to your healthcare provider to see if Paragard is right for you.

Paragard is a hormone-free IUD (intrauterine device) that prevents pregnancy for up to 10 years using copper.

Important Safety Information

- Don't use Paragard if you are or may be pregnant, have fibroids, a pelvic infection including pelvic inflammatory disease (PID), get infections easily, certain cancers, unexplained bleeding, Wilson's disease, or a copper allergy. IUDs, including Paragard, have been associated with an increased risk of PID.

Please see additional Important Safety Information throughout this brochure and accompanying Full Prescribing Information.

What Is Paragard®?

Paragard is a small IUD (intrauterine device) that uses just 1 simple active ingredient to prevent pregnancy over 99% of the time.

It's placed in your uterus by a healthcare provider during a routine office visit. Once in place, it provides continuous pregnancy prevention for as short or long as you want—up to 10 years.

Paragard is made mostly of soft, flexible plastic wrapped with a thin layer of copper around the arms and stem.



Important Safety Information (continued)

- Pregnancy with Paragard is rare but can be life threatening and cause infertility or loss of pregnancy.
- Paragard may attach to or go through the uterus and cause other problems.
- Tell your healthcare provider (HCP) if you develop severe pain or fever shortly after placement, miss a period, have abdominal pain, or if Paragard comes out. If it comes out, use backup birth control.
- Tell your HCP you have Paragard before having an MRI or a medical procedure using heat therapy.
- At first, periods may become heavier and longer with spotting in between.

IUD (intrauterine device) = IUS (intrauterine system)

*Must be removed by a healthcare provider.

†Check with your insurance provider to confirm coverage related to your individual plan. Fees for the Paragard placement procedure may apply.

Why Paragard®?



100% hormone free



Over 99% effective



Works using just **1 simple active ingredient**—copper



No hassle or daily birth control routine—just a monthly thread check



Lasts up to 10 years but can be removed anytime sooner if you decide you want to get pregnant*



After removal, the effects of Paragard are **immediately reversible**



Non-surgical placement during a routine office visit



The only IUD **FDA approved for over 30 years**



Most women with insurance can get Paragard at **no cost**†

Important Safety Information (continued)

- Additional common side effects include anemia, pain during sex, backache, and vaginal discharge.
- **Paragard does not protect against HIV or STDs.**

Please see additional Important Safety Information throughout this brochure and accompanying Full Prescribing Information.

 **Paragard**[®]
intrauterine copper contraceptive
simple, honest
pregnancy prevention*

 **Paragard**[®]
T380A
intrauterine copper contraceptive

simple,
honest
pregnancy
prevention™

**FOR
SEX
SAKE.***



Ask your healthcare
provider for Paragard[®].

Scan the QR code with your
phone's camera to visit
Paragard.com

Visit [Paragard.com](https://www.paragard.com)
or call 1-877-PARAGARD.



***Paragard does not protect against HIV or STDs,
and does not enhance sexual experience.**

Important Safety Information (continued)

Only you and your HCP can decide if Paragard is
right for you. Available by prescription only.

You are encouraged to report negative side
effects of prescription drugs to the FDA at
www.fda.gov/medwatch or call 1-800-FDA-1088.

**Please see additional Important Safety Information
throughout this brochure and accompanying Full
Prescribing Information.**

Paragard[®] is a registered trademark of CooperSurgical, Inc.
© 2026 CooperSurgical, Inc. All Rights Reserved.
C-US-PAR-000918 January 2026